

# CHEVRON®

## USER GUIDE

CHEVRON® TOILET SURROUND

ORDERING CODE: 221



SAFE WORKING LOAD 130KG

## 1. INTENDED USE

The toilet surround is manufactured from pre-treated anti-rust steel. The height adjustable surround is designed to fit most toilets, comes with anti-slip rubber tips for safety and is intended to be used over a toilet to aid in sitting and standing.

## 2. SAFE USE



**WARNING!** This user manual must be read before using the toilet surround. Severe injury or death may result if user instructions, maintenance instructions and product warnings are not followed.



**WARNING!** The decision to use this toilet surround should always be guided by a qualified Health Care Professional, who can also advise and assist with the correct height adjustment and usage instruction.

- Do not exceed the safe working load of 130kg.
- Check that the toilet surround is firmly assembled before use.
- Inspect at regular intervals for wear and damage. Remove from use immediately if faults are found.
- Supervision or assistance when using the toilet surround is recommended for users with limited mobility.
- Use the toilet surround only for the purpose it is intended for.

## 3. PRODUCT SPECIFICATIONS

Safe working load	130kg
Width inside arms	450mm
Armrest height adjusts	660mm – 790mm

### OPTIONS



## 4. INSTRUCTIONS FOR USE


### ADJUSTING THE HEIGHT

The surround is height adjustable to match individual user requirements. A series of holes along each leg allow armrest height to be adjusted from 660mm to 790mm. A brass locking button is used to secure the height of each leg.


Press in the locking button to disengage, and turn the leg slightly so the button is no longer aligned with the holes. Move the leg up or down to your ideal height setting, then turn the leg back to align the locking button with the nearest hole. This is applicable to 3 out of the 4 legs. The remaining leg has extra holes which can be adjusted to help stabilise the toilet surround if the floor is uneven.

The locking button must be fully engaged and protruding through the height adjustment hole to ensure a positive lock.

Before use, test that locks are fully engaged and ensure the surround is stable and resting firmly on the floor's surface.

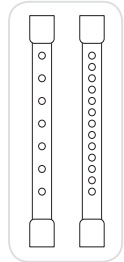


**WARNING!** Ensure all 4 legs are set so that the toilet surround is level. Severe injury or death may result if this is not done correctly.

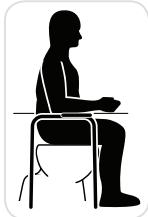

### IDEAL ARMREST HEIGHT

With the user seated, ideal armrest height is at elbow height, with user's arms resting against their sides and lower arms resting flat on armrest.

### HOW TO USE THE TOILET SURROUND

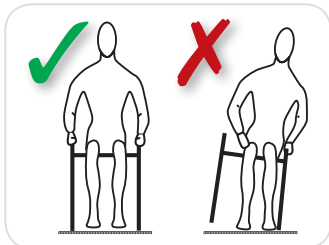

All four legs must be level on the floor and cannot slide.

- To sit on the toilet, move backwards until you can feel the seat behind your legs, place each hand on the armrest of the toilet surround. Support your body and prevent the toilet surround from slipping away. Then lower yourself down.
- When getting off the toilet, move your bottom to the front edge of the seat ensuring your feet are firmly placed on the floor. Use the armrests to push yourself into a standing position leading with your head and shoulders.